

»Initial results show a reduction in pain and indicated an improvement in gait and a positive influence on proprioception of the osteoarthritic knee.

The innovative design makes it comfortable for the patient to wear, making them more likely to persevere with treatment.«



Prof. Dr. med. Stefan Sell — Medical Director
Gelenkzentrum Schwarzwald Enzkreisliniken Neuenbürg

STABILITY, MOBILITY, AND PAIN PERCEPTION BEFORE AND AFTER TREATMENT WITH GENUTRAIN OA



● Before treatment
● After treatment

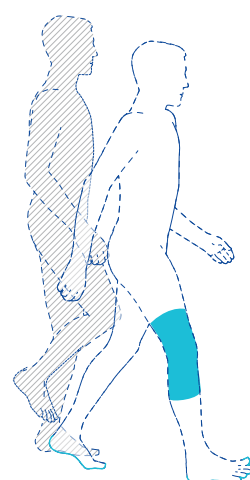


Abb.: Pain, stability, mobility, and pain-free walking distance before treatment with the orthosis and after treatment with the orthosis (an average of 5.4 weeks), specified using a visual analog scale. Showing averages with standard deviation; * p < 0.001 (paired t-test)

Source: Non-interventional, clinical study; case series, one-arm (evidence level 3), „Evaluation of effect and therapeutic safety of the GenuTrain® OA knee orthosis“, Bauerfeind AG

GenuTrain® OA IN THE BAUERFEIND OSTEOARTHRITIS OF THE KNEE PORTFOLIO



GenuTrain® A3
Level I – III

Mild osteoarthritis



GenuTrain® S
Level I – III

Mild osteoarthritis with joint instability



GenuTrain® OA
Level II – III

Moderate osteoarthritis with varus or valgus misalignment



SecuTec® OA
Level III – IV

Moderate to severe osteoarthritis with varus or valgus misalignment

SIZE CHART CHOOSE FROM FIVE STANDARD SIZES:

Size	1	2	3	4	5
Circumference top (cm)	40 – 47	47 – 52	52 – 58	57 – 65	60 – 72
Circumference top (inch)	15¾ – 18½	18½ – 20½	20½ – 22¾	22½ – 25½	23½ – 28¼
Circumference bottom (cm)	28 – 34	34 – 39	39 – 43	43 – 46	46 – 50
Circumference bottom (inch)	11 – 13½	13½ – 15¼	15¼ – 17	17 – 18	18 – 19¾

Order number 121 4165 0070 +0°
+6 = right for medial relief; or lateral relief of the left leg, +7 = left for medial relief; or lateral relief of the right leg; ° = size (1–6)

Optional: Restriction set: 129 4166 0070 00*
*1 = for sizes 1 and 2, *3 = for sizes 3 and 4, *5 = for size 5



Disclaimer | All information given in this brochure is up to date at the day of first publishing. The brochure allows – depending on the information given – an initial overview and suggests some, but not all options of medical diagnosis and treatments to experts only. This brochure does not address patients. Although scientifically based, the brochure by no means claims to be complete or show all existing doctrinal. The entire content of this brochure – also in extracts, no matter how published, and especially concerning diagnostics and forms of treatment (in text, pictures, graphics, images or other material) has to be understood as non-binding support for the therapy to be selected finally by medical doctor, physician or other medical professionals. The brochure's intension is not to offer the full range of all medical aspects and latest scientific discussion that should be mandatorily considered before taking actions. We strongly recommend individual medical advice. If in doubt, please seek additional opinion of medical experts, who we would be delighted to provide.

Bauerfeind AG
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany

P +49 (0) 800 0 01 05 20
F +49 (0) 800 0 01 05 25
E info@bauerfeind.com

Distributed by

JOINT OPERATIONS LIMITED
Unit 11 Interface Business Park
Bincknoll Lane
Royal Wootton Bassett
Wiltshire
SN4 8SY



GENUTRAIN® OA

ACTIVE WITH OSTEOARTHRITIS





TARGETED RELIEF AND STABILISATION FOR MORE ACTIVITY WITH KNEE OSTEOARTHRITIS

INDICATIONS

- Medial or lateral knee osteoarthritis
- Instability (mediolateral)
- Unicompartmental cartilage/bone lesion
- Condition after cartilage surgery and cartilage repair for necessary unilateral post-operative relief
- Following unilateral meniscal lesions
- Ahlbäck's disease (osteonecrosis)
- Osteochondrosis dissecans
- Bone bruising (bone edema of the medial condylus of the femur/tibial plateau)

RELIEF SYSTEM TO INCREASE THERAPY ADHERENCE

- **Threepoint Principle**

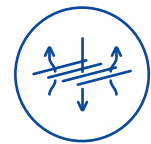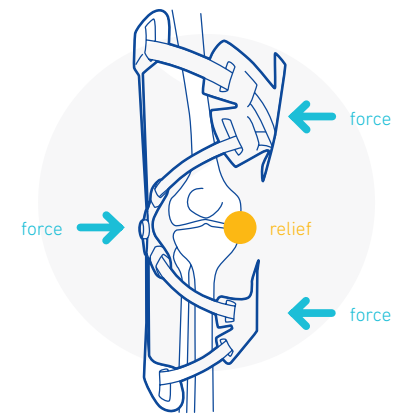

ACTIVE COMFORT



PERFECT ANATOMIC FIT



CONTROLLED ADJUST

GenuTrain® OA



**THREE-
POINT
PRINCIPLE**

**BOA®
BOA® FIT
SYSTEM**

**GENUTRAIN® OA
INCREASED
EFFICIENCY FOR
YOUR MEDICAL
SUPPLY RETAILER**

- + Reducing pain
- + Increasing activity
- + Optimizing compliance
- + space-saving storage
- + tool-free moldable hinge
- + patient-friendly readjustment

INNOVATIVE DESIGN FOR AN ACTIVE APPROACH TO TREATMENT

HIGH LEVEL OF ACCEPTANCE FOR LONG PERIODS OF USE

- Pressure-free bending zone in the hollow of the knee
- Non-slip design
- Breathable 3D knitted mesh
- Flexible hinge on the side
- Lightweight



EFFECTIVE RELIEF

- Relief stabilising system with:
- Three-point principle
 - Surrounding bands
 - Flexible hinge



EASY HANDLING

- Easy to put on and take off thanks to large donning and removal aids and flexible knitted fabric
- Individually adjustable relief thanks to **Boa® Fit System**



PUSH IN TO
ENGAGE



TURN TO
TIGHTEN



PULL UP
FOR QUICK
RELEASE

